Product description

The EvacPRO is a flexible lightweight compact stretcher system for use by the emergency services, to evacuate immobile people from such places as nursing homes, hospitals, airplanes, trains, pubs, clubs, schools and hotels in the event of a medical emergency or other emergency situation where the use of conventional patient transfer equipment would not work.

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The EvacPRO is designed to facilitate the manual movement of immobile patients by maximising the use of sliding techniques where possible and minimising the impact on the personnel performing the movement through the use of good ergonomic design features.

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The EvacPRO is designed to stabilise and hold the patient securely whilst being moved but still have a good degree of flexibility to negotiate obstacles along the transfer route. If required extra stability and support can be added by inserting a standard medical long board into the long board pocket on the upper surface of the EvacPRO. This can be performed either with or without the patient in the EvacPRO, from either the foot or head end via the Velcro openings in the pocket.

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Specifications

Packed size:	55cm x 30cm x 30cm weight: 11kg
Deployed size:	206cm x 152cm x 2cm
Safety tested to:	500kg
Working Load Limit (WLL):	400kg
Material Construction:	P300 Plastic base (black 2mm board is replaceable), heavy duty PVC upper, 50mm colour coded safety webbings fitted with metal seatbelt type adjustable buckles.

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Testing & Cleaning

The EvacPRO should be inspected regularly for any signs of damage or excessive wear, the black 2mm plastic base can be removed and replaced if damaged, any excessive wear or damage to the main structure should be referred back to the manufacturer. If wiped or washed clean, any detergents or disinfectants should be well rinsed off and the EvacPRO dried before being replaced in its rolled storage position inside the foot pocket.

Additional information:

Contact: info@jolysignal.be

Visit: EVACPRO

Tél. : +32(0)470/540.185 - Fax : +32(0)65/632.054

Rue de Carochette, 3 A (B)7380 BAISIEUX

info@jolysignal.be - www.jolysignal.be